



**For patient: please keep this form until receipt of your results by your Doctor.**

The ImmunoProfile requires the blood samples to be collected only on the bottles provided within this specific kit **ImmunoBox**.  
Your doctor can order it for you by phone at **+33 4 92 00 43 20** or by email **immunoprofil@labo-barla.eu**.

Thank you for choosing BARLA Laboratory for your tests.

## Sample Taking Instructions

**Please collect blood samples on Monday or Tuesday ONLY for same-day shipping**

Please do not collect the day before a French Bank Holiday

French Bank Holidays in 2017: 17-April, 1st-May, 8-May, 25-May, 14-July, 15-August, 1st-November, 11-November, 25-December, 1-January 2018

- Please collect: - **All red top bottles** (with gel for serum separation)  
- **All purple top bottles** (EDTA)
- **Please fill bottles to appropriate level** and identify them with name and surname.
- Please follow instructions stated on plastified sheet placed in the ImmunoBox.
- Please put back the transparent plastic cover after placing full bottles into the ImmunoBox .
- Please fill in and sign "Blood draw form".

*Name and date of birth fields are mandatory!*

*The samples that have not been delivered to the Laboratory within 24 hours after collection might be damaged and unfit for further studies.*

## Sample Taking Instructions for Borrelia EliSpot

**Please collect blood samples on Monday or Tuesday ONLY for same-day shipping.**

Please do not collect the day and two days before a French Bank Holiday

French Bank Holidays in 2017: 17-April, 1st-May, 8-May, 25-May, 14-July, 15-August, 1st-November, 11-November, 25-December, 1-January 2018

- The patient don't have received treatment like : antibiotics, immuno-suppressive or immuno-moludator (corticoid). In case of control test after antibiotics course, please respect a three-month intervale after completion of treatment course.
- Please collect two light blue top bottles (citrate).
- **Please fill the bottles to appropriate level** and fill in full patient's name.
- Please fill the blood draw form.
- In case of shipping from the patient's address, please inform the patient that the samples must be stored until shipping

## Instructions for sender

### Preparing ImmunoBox on the day of shipping

#### Enclose following documents into the ImmuBox containg blood bottles :

- Signed Clinico-Biological Information Form. Make sure the current treatments (if relevant) are specified (name of the medication, date).
- Signed Blood Draw Form.
- Request Form/Prescription signed by your Doctor.

#### Please close the ImmunoBox properly.

- This shipping kit is compliant with European Regulation on transportation of infectious biological substances.

#### Please place the ImmunoBox in the DHL plastic bag provided.

### Instructions for shipping by DHL on the day of the blood draw

#### On the day of blood draw in the morning:

- Call **DHL at 0844 248 0844** to book a collection using following **account number: 956 896 131**.
- Fill in the DHL waybill provided (Paragraph 2: name, adress of the sender, phono numbers) and paragraph 7: date and signature).
- Place the DHL waybill in the external pocket on the DHL bag.

*The samples that have not been delivered to the Laboratory within 24 hours after collection might be damaged and unfit for further studies!*

## ImmunoProfile: Pathology report and billing information

### The results and the invoice will be posted to your doctor.

Shipment fees of 45 EUR will be added to the cost of the lab test(s).

Your doctor can select the preferred way of transmission of the results within options quoted above. Upon request the results can be securely emailed. The login details can be also provided for access the results via our digital plateform [www.labo-barla.eu](http://www.labo-barla.eu).